

PARASSITO KIT (MIF)

IVD in Class A, EU Reg. 2017/746

For in vitro diagnostic use

Colouring and fixative solution for eggs and faecal parasite.

DESCRIPTION

Colouring and fixative solution for eggs and faecal parasite.

KIT CONTENTS

R1. MIF Solution Container **40 pcs x 13 mL**
MIF Solution:

 Merthiolate
 Formalin
 Eosin
 Glycerol

R2. Lugol Solution **2x20 mL**
bottles

 Iodine
 Potassium Iodide

Graduated Pasteur pipettes **40 pcs**

WARNING AND PRECAUTIONS

For in vitro diagnostic use.

Observe the precautions normally taken when handling laboratory reagents.

Safety Data Sheet is available on request for professional users.

All wast must be disposed of according to local directives.

STORAGE AND STABILITY

PARASSITO KIT (MIF) **10-35°C**

PARASSITO KIT (MIF) is stable until the expiration date indicated on the label under the recommended storage conditions.

INSTRUCTION FOR USE

- Take 1 mL of Lugol Solution and transfer it into the MIF Solution Container;
- Take with apposite loop a little quantity of faeces and crush it against the internal side of flask;
- Close up the lid and mix until the homogenous suspension of sample;
- Leave to replace for 1 hour;
- Take with a pipette a little quantity of sample and to put down a drop on microscope slide;
- Cover the microscope slide and examine at microscope.

INSTRUCTION FOR FAECAL SAMPLES COLLECTION

- The patient must be suspend any teraphy with antiacid, antidiarroic, antimicrobial and laxative drug.
- The faecal samples must be collected in clean, large container or on clean paper sheet; do not utilize faeces from toiolettes or mixing with urine.
- Send the sample to laboratory within 1 hour to emission.

REFERENCES

1. J.W. Smith and M.S. Bartlett. "Diagnostic parasitology. Introduction and methods", 55 p.595-611. In E.H. Lennette, A. Balows, W.J. Hausler J.r., H.J. Shadomy (ed) Manual of Clinical Microbiology, Fourth Edition American Society for Microbiology, Washington,D.C. 1985.
2. D.M. Melvin and G.R. Healy "Intestinal and urogenital protozoa", 57, p.631-350. In E.H. Lennette, A. Balows, W.J. Hausler J.r., H.J. Shadomy (ed) Manual of Clinical Microbiology, Fourth Edition American Society for Microbiology, Washington,D.C. 1985.
3. L.R. Ash and T.C. Orihel "Intestinal helminths", 59, p. 660-673. In E.H. Lennette, A. Balows, W.J. Hausler J.r., H.J. Shadomy (ed) Manual of Clinical Microbiology, Fourth Edition American Society for Microbiology, Washington,D.C. 1985.

PRESENTATION

Packaging
REF.
PARASSITO KIT (MIF)
40 Test
8052

SYMBOLS



Read the instructions



Biological hazard



CE Mark (product complies with the requirements of Regulation (EU) 746/2017)



Temperature limitation



Use by



For in vitro diagnostic use



Manufacturer